# Safe-Wrap™

IVD

For In Vitro  
Diagnostic  
Use

# Blood Collection Tubes

REF CT 220

## ENGLISH

### INTENDED USE

RNA Medical® Safe-Wrap Blood Collection Tubes are used for the collection of capillary blood samples for the measurement of pH, blood gases, and electrolytes (excluding lithium). This product should not be used for the collection of samples for coagulation or immunoassay measurement and should not be used to transfer aqueous control samples.

### PRODUCT DESCRIPTION

Safe-Wrap Blood Collection Tubes are Mylar®-wrapped glass capillary blood collection devices that have been treated with calcium-balanced lithium heparin to prevent the sample from clotting. The Mylar wrapping serves to minimize the risks associated with broken glass capillaries, including glass cuts and aerosol formation, by containing both the glass and the sample in the event of accidental breakage. Each tube can hold up to 220 µL and has line markings at 55, 110 and 165 µL (approximately) to allow for flexibility of the sample volume collected. Safe-Wrap Blood Collection Tubes are packaged in cylinders containing 100 tubes.

#### Active Ingredients:

Safe-Wrap Blood Collection Tubes are treated with calcium-balanced lithium heparin (derived from porcine intestinal mucosa) at a concentration of approximately 70 IU/mL of whole blood sample.

**Caution:** Practice the recommended "Universal Precautions" when handling and using this device. Discard as biohazardous waste after use.

### STORAGE

The expiration date stated on the Safe-Wrap Blood Collection Tubes packaging is for product stored at temperatures of 2-30 °C.

### DIRECTIONS FOR USE

Follow the policies and procedures for capillary blood collection documented by your facility. Suggestions for sample collection and handling are listed below.

Fill the tube to the approximate volume needed as indicated by the reference fill marks in Figure 1. When filling the tube from either end, the first line represents 55 µL, the second 110 µL, and the third 165 µL. A completely filled tube will hold approximately 220 µL. Once the tube is filled to the desired volume, mix the sample by holding the center of the filled tube horizontally between thumb and forefinger and rolling gently. Analyze the blood sample within 10-15 minutes of collection.

#### Consider the following for sample collection and handling:

- When collecting samples for pH and blood gas analysis, the puncture site should be warmed to increase blood flow.
- Allow alcohol applied to puncture site to dry before performing the puncture to avoid hemolysis.
- Wipe away the first drop of blood to avoid contamination with tissue fluid.
- Collect blood from the center of the blood drop after it forms.
- Avoid squeezing the puncture site excessively during collection as hemolysis may occur.
- Do not use the capillary tube if an air bubble has been trapped in the sample. This may result in erroneous pO<sub>2</sub> results.
- Do not use the sample if it clots or if the glass breaks.

### OPTIONAL ACCESSORIES

RNA Medical offers End Caps, Cat. # EC 220, which are capillary tube closure devices specifically sized for use with Safe-Wrap Blood Collection Tubes. Please contact RNA Medical for further information.

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Safe-Wrap is a trademark of Bionostics, Inc., Devens, MA, USA. RNA Medical is a registered trademark of Bionostics, Inc., Devens, MA, USA. Mylar is a registered trademark of E. I. DuPont, Inc., Wilmington, DE, USA.
RNA Medical, Division of Bionostics, Inc.