



February 2, 2026

Dear Valued Customer,

CVC223 set lot 522345, Expiration date 2026-11-30 has incorrect target values and ranges for the following Hemoglobin targets and ranges.

Avoximeter 1000E, Level 1 O2Hb, Lot 55142, Expiration date 2026-11-30

	Revised Values O2Hb %				Original Values O2Hb %		
Analyte	Mean	Low	High		Mean	Low	High
O2Hb	35.3	30.7	39.9		36.2	31.6	40.8

Avoximeter 4000, Level 2 O2Hb and COHb, Lot 54881, Expiration date 2026-12-31

	Revised Values O2Hb and COHb %				Original Values O2Hb and COHb %		
Analyte	Mean	Low	High		Mean	Low	High
O2Hb	98.1	93.6	102.6		98.9	94.4	103.4
COHb	3.4	-0.6	7.4		2.0	-2.0	6.0

Avoximeter 4000, Level 3 O2Hb and COHb, Lot 54978, Expiration Date 2026-12-31

	Revised Values O2Hb and COHb %				Original Values O2Hb and COHb %		
Analyte	Mean	Low	High		Mean	Low	High
O2Hb	83.4	79.1	87.7		84.5	80.2	88.8
COHb	16.4	11.9	20.9		15.5	11.0	20.0

Avoximeter 4000, Level 4 O2H and COHb, Lot 55075, Expiration Date 2027-01-31

	Revised Values O2Hb and COHb %				Original Values O2Hb and COHb %		
Analyte	Mean	Low	High		Mean	Low	High
O2Hb	55.6	51.3	59.9		56.9	52.6	61.2
COHb	43.6	38.3	48.9		43.2	37.9	48.5

The insert that comes with the kit will not contain this information. This letter will serve as notification for the value assignment of these analytes for this lot. Please keep this announcement with your records for as long as you have this lot in stock.

If you have any questions regarding this notification, please feel free to call or email us using the contact information provided.

RNA Medical, Division of Bionostics, Inc.
7 Jackson Road
Devens, MA 01434
Phone: 978-772-9070 / 800-533-6162
E-Mail: customerservice@rnamedical.com

BIOMKT-242 Rev. A

RNA Medical, Division of Bionostics, Inc
7 Jackson Rd Devens, MA 01434-4204 USA ▪ Toll-free: (800) 533-6162 ▪ Phone: (978) 772-9070
Fax: (978) 964-0537 www.RNAMEDICAL.com