



April 10, 2018

A review of our QC253 level 1, Full Range CO-Oximeter Control, lot# 74856 with an expiration date of 2019-02, has prompted us to adjust our assigned value for the AVOXimeter® 1000E and AVOXimeter® 4000 analyzers. Please see the chart below for the corrected values.

Expected Values Chart

---

<b>Analyzers</b>	<i>mean</i>	<i>range</i>	<i>mean</i>	<i>range</i>	<i>mean</i>	<i>range</i>	<i>mean</i>	<i>range</i>
	<b>tHb</b>		<b>O2Hb</b>		<b>COHb</b>		<b>MetHb</b>	
<b>Accriva</b>	<b>g/dL</b>		<b>%</b>		<b>%</b>		<b>%</b>	
AVOXimeter 1000E	7.6	6.9 - 8.3	93.1	88.6 - 97.6				
AVOXimeter 4000	7.6	6.9 - 8.3	94.9	90.4 - 99.4	5.5	1.5 - 9.5	-0.1	-3.0 - 2.8

RNA Medical has not reprinted the insert for this lot, so please use this letter as notification for this change. Keep this letter with your records as long as you have this lot in stock. This letter will also be available on our website.

Please do not hesitate to contact RNA Medical at 1-800-533-6162 or by email at [CustomerService@RNAMedical.com](mailto:CustomerService@RNAMedical.com) if you have any questions regarding this revision.

BIOMKT-194  
REV. A